Skater™ FIX

ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com


EMERGO EUROPE
Westervoortsejdijk 60
6827 AT Arnhem
The Netherlands
+31 70 345 8570

Emergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

Skater Fix Direction of Use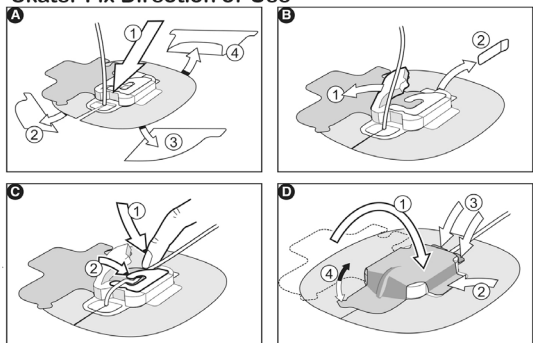

X9585767

Date of Issue: 08/14/2023 Rev. B

<https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Bulgarian	Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols
Czech	Slovník symbolů v elektronické formě je umístěn na www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
German	Das Glossar der Symbole kann unter www.argonmedical.com/symbols abgerufen werden
Greek	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse en línea en la dirección www.argonmedical.com/symbols
Estonian	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols
Finnish	Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols .
French	Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols
Hungarian	A szimbólumok összefoglalása elektronikusan itt található: www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Lithuanian	Simbolių žodyno elektroninė forma pateikta www.argonmedical.com/symbols
Norwegian	Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Polish	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols
Romanian	Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols
Slovak	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols

Slovenian	Slovar simbolov se nahaja v elektronski obliki na www.argonmedical.com/symbols .
Swedish	Symbolforklaringen hittas elektroniskt på www.argonmedical.com/symbols
Turkish	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır

en - English

Intended Use/Purpose

SKATER™ FIX is a catheter fixation device designed to secure a drainage catheters.

Device Description

Skater™ FIX is a latex-free device, consist of central thin carrier that has hydrocolloid adhesive and a covered with non-woven adhesive medical grade. The soft foam support block and velcro prevent the catheter from kinking. The velcro cover can be opened for puncture site allowing for inspection for infection or leakage.

Indication for use

The Skater™ FIX is indicated for the fixation of drainage catheters.

Contra-indications for Use

There are no contraindications with use of this product. However, the assessment of the patient's overall physical state by the physician is required prior to the procedure.

Possible complications/Risks:

There no known complications for this device.

Warnings

- Contents are supplied sterile and are intended for single use only. Do not re-sterilize.
- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Do not use if package is open or damaged and if the expiration date has passed.

Precautions

- The product must only be used by licensed physician.
- Mechanical forces are minimal and under control of user. Risk acceptable if used as intended by license physician.

Disposal

After use, handle and dispose in accordance with hospitals policies and procedures concerning biohazard materials and waste.

Storage: Store at controlled room temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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